

LOCATING PATIENTS

Use the *Performed*, *Status*, or choose from the other search criteria to locate the desired patients. For example, you can use the status of the exam and the patient's last name to narrow down your search. Click **GO** to start your search.


My Search – Runs your saved default search.


Clear – Erases all the search fields.


Save – Saves your current search criteria as the default search.

Query Archive – Queries the Archive and initiates transfers from the Archive to the Web server. This is an optional feature.

VIEWING REPORTS AND STUDIES

Click the View Report  icon to view an RIS report.

Click the View First Impression  icon to view a First Impression text report.

Click the Listen To Report  icon to hear a First Impression audio report.

If the Exam has prior studies, click the plus sign to expand the view and list prior studies.

In the Exam List, if the patient's name is underlined, click the patient's name to view the available study. The other underlined fields allow the values to be changed. This is a feature that can be enabled for selected users and disabled for others

GETTING STARTED

Step 1. Start your Internet Explorer Web browser. Enter the server name in the Address field.

Step 2. Enter your username and password, then click Login.

Netscape users:

When the Java Security window appears, select **Grant** to allow the viewer software to be installed.

When "Install Successfully Complete" appears, click **OK** to continue.

Internet Explorer users:

Click **Yes** to accept the viewer software from eMed Technologies.

- ▶ **Disable All Pop-Up Blockers**
- ▶ **Allow ActiveX Java Script To Load**





Opens the Exam List.



Opens a window that displays any reports for this patient.



Navigates between series. The first number, 10, is the series number. The number in parenthesis (23) is the number of images in the series.



Navigates to the next or previous image.



Opens the First Impression/Assign tool.



Brings up the Image Printing Screen.

Minimum System Requirements

- 350MHz or faster processor
- 256MB RAM
- 200 MB of free disk space
- 24 or 32 Bit True Color
- 1024 x 768 Display Resolution
- DSL, Cable or high-speed network connection
- Internet Explorer 5.5 SP2 or later
- Windows 98, Me, XP, NT, or 2000



Switches between Tile and Stack modes.



Selects the W/L adjustment tool. Click on the arrow to select from W/L presets.



Selects the Scroll tool. Only available in Stack mode and in a 1x1 layout in Tile mode.



Selects the Magnify Glass tool. Click on the drop-down arrow to select different magnifying glass sizes.



Activates the pan tool. You must apply a zoom for the pan tool to work.



Activates the Variable Zoom tool. Click the drop-down arrow for discrete zoom factors.



Activates the measure tool. Click the drop-down arrow to select different measuring options.



Changes the amount of text displayed: None, Basic, or All.



Changes the number of images on the screen at a given time.



Reorders the images within a series.



Displays a cross-reference line on an image. Only available in Stack mode.



Links and unlinks the viewports in Stack mode.



Automatically scrolls the images in a viewport. Only available in Stack mode.



Tools to rotate, flip, and mirror images. Also has a grayscale invert mode.



Applies a filter to edge-enhance the image.